**F&P** Evora

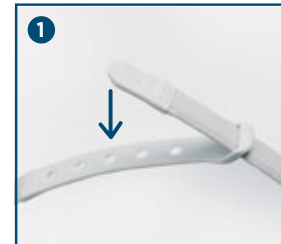
## Nasal Mask

### Use and Care Guide

**F&P** Evora


## FITTING YOUR MASK

## F&P Evora Use and Care Guide

Evora is a trademark of Fisher & Paykel Healthcare Limited.  
For patent information, see [www.fphcare.com/ip](http://www.fphcare.com/ip)

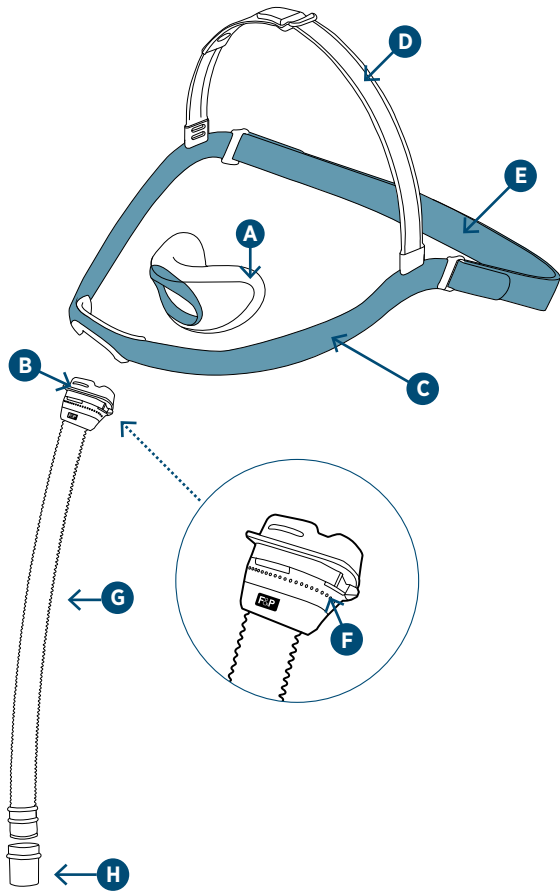REF 900183 REV A 2019-06 © 2019 Fisher & Paykel Healthcare Limited

PAGE	LANGUAGE
1	English
8	Français (French)
15	Español (Spanish)

## INTENDED USE/ INDICATIONS FOR USE

The F&P Evora™ Nasal Mask is intended to be used by adults weighing  $\geq 66$ lbs (30kgs) who have been prescribed non-invasive positive airway pressure therapy such as CPAP or bilevel by a physician. The F&P Evora Nasal Mask is intended for single patient use in the home.

## OPERATING INSTRUCTIONS

- The operating pressure range of your mask is 4 to 25 cmH<sub>2</sub>O.
- The operating temperature range of your mask is 5 to 40°C (41 to 104 °F).
- Before using your mask each time, you should:
  - i. Inspect it for damage or deterioration. If there is any visible deterioration (cracking, tears, etc.), do not use your mask and seek replacement part(s) from your healthcare provider.
  - ii. Inspect the Exhaust Holes **(F)** on the Frame **(B)**. Ensure air is flowing through these holes and do not use if blocked.
  - iii. Make sure the air path through the mask is free from objects that may prevent breathing.

**Note:** Please refer to the Warnings and Cautions  within this guide. Failure to follow the operating instructions above may compromise the performance and safety of the mask.

## MASK PARTS


**Refer to the mask diagram on the inside front cover of this guide.**

- |                      |                          |
|----------------------|--------------------------|
| <b>(A)</b> Cushion   | <b>(E)</b> Back Strap    |
| <b>(B)</b> Frame     | <b>(F)</b> Exhaust Holes |
| <b>(C)</b> Headgear  | <b>(G)</b> Tube          |
| <b>(D)</b> Top Strap | <b>(H)</b> Swivel        |

**Note:** The Tube **(G)** is permanently connected to the Frame **(B)**.

## FITTING YOUR MASK

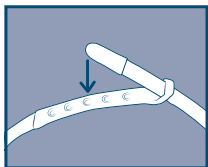
Remove all packaging before using your mask. Refer to the mask diagram on the inside cover to identify the mask parts.

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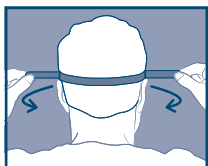2. Hold the front of the mask and pull the Back Strap **(E)** over the back of the head. Pull the mask down to position the Cushion **(A)** under your nose.

3. Adjust Back Strap **(E)** to tighten or loosen the Cushion **(A)** against your face by pulling evenly on the tabs.

4. Adjust the Top Strap **(D)** button to tighten or loosen the Headgear **(C)** so that it is sitting just above your ear without touching it. You may need to take the mask off to do this.

The Top Strap **(D)** should sit just forward of the middle of your head.

5. Connect the Swivel **(H)** to the CPAP hose and with the CPAP on, readjust the Back Strap **(E)** until the mask seals and the Cushion **(A)** feels comfortable under your nose.

## USEFUL TIPS

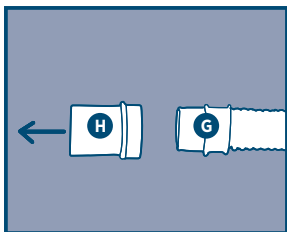
- Do not overtighten the mask.
- Your nose should never be inside the Cushion **(A)** opening.
- If you are experiencing any leaks try repositioning the Cushion **(A)** under your nose.

## USING YOUR MASK


To keep your adjustments, it is not necessary to undo the Top Strap **(D)** or Back Strap **(E)** each time you remove and refit your mask.

To remove your mask, simply hold the front of the mask, pull forward away from your nose and lift the mask up and over your head.


## DISASSEMBLING YOUR MASK

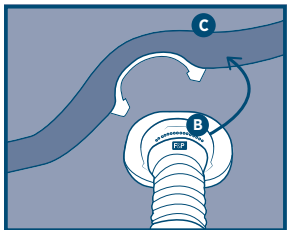1. Pull the Swivel **(H)** from the bottom of the Tube **(G)**.

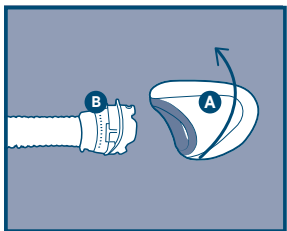2. Unclip the front of the Headgear **(C)** from the Frame **(B)** located at the top of the Tube **(G)**.

3. Pull the bottom of the Cushion **(A)** to unclip from the Frame **(B)**.

## CLEANING YOUR MASK AT HOME

- **After each use:** Wash the Cushion (A), Swivel (H), Frame (B) and Tube (G).
  - **After 7 days use:** Wash the complete Headgear (C) including the Top Strap (D) and Back Strap (E). It is not necessary to remove the Back Strap (E) when cleaning the Headgear (C).
1. Hand-wash the mask parts in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
  2. Rinse thoroughly in fresh water, ensuring that all soap residue has been removed.
  3. Inspect all parts to ensure they are visually clean. If necessary, repeat the cleaning steps until all parts are visually clean.
  4. Leave all parts to dry out of direct sunlight before reassembling.

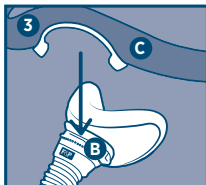
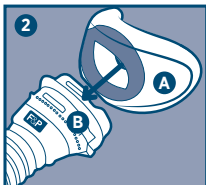
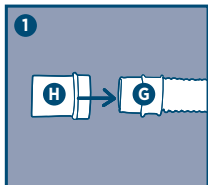
### ⚠ Cleaning Cautions

- Do not clean your mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer.
- Do not clean your mask in a dishwasher.
- Do not store your mask in direct sunlight.
- Do not iron the Headgear (C).
- Do not pull or attempt to remove the Tube (G) from the Frame (B). The Tube (G) is permanently connected to the Frame (B).


**The above actions may deteriorate or damage the mask and shorten its life. If there is any visible deterioration do not use your mask and seek replacement part(s).**
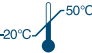
## ASSEMBLING YOUR MASK

1. Push the Swivel (H) onto the bottom of the Tube (G).
2. Push the Cushion (A) firmly onto the Frame (B) to click into place.
3. Push the Headgear (C) down onto the Frame (B) behind the Exhaust Holes (F) to click into place.

## SYMBOL DEFINITIONS

	Catalogue number	CE 0123	Conforms with medical device directive 93/42/EEC
	Manufacturer	<b>Rx only</b>	Prescription only
	Authorized European Representative		Small seal size
	Batch code		Medium seal size
	Storage Temperature limitation		Large seal size
	Date of manufacture		Wide seal
	Consult instructions for use		Date of shelf life expiry
	Caution		

## OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (REF 900HC452). Please contact your healthcare provider.

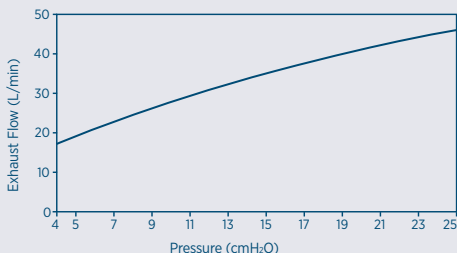
## TECHNICAL SPECIFICATIONS

The F&P Evora Nasal mask has exhaust holes to expel the air that you exhale from the mask. It is important that these exhaust holes are not blocked by any object. This controlled leak ensures all exhaled CO<sub>2</sub> is expelled from the mask.

### F&P EVORA EXHAUST FLOW INFORMATION

Pressure (cmH <sub>2</sub> O)	4	5	7	9	11	13	15	17	19	21	23	25
Flow (L/min)	17	19	23	26	29	32	35	38	40	42	44	46

Due to manufacturing variations, the exhaust flow rates may vary from the nominal values shown above.

### RESISTANCE TO FLOW

Pressure drop through mask with small Cushion at 50 L/min: 1.0 ± 0.1 cmH<sub>2</sub>O

Pressure drop through mask with small Cushion at 100 L/min: 1.4 ± 0.25 cmH<sub>2</sub>O

Pressure drop through mask with medium Cushion at 50 L/min: 1.0 ± 0.1 cmH<sub>2</sub>O

Pressure drop through mask with medium Cushion at 100 L/min: 1.2 ± 0.25 cmH<sub>2</sub>O

Pressure drop through mask with large Cushion at 50 L/min: 1.0 ± 0.1 cmH<sub>2</sub>O

Pressure drop through mask with large Cushion at 100 L/min: 1.2 ± 0.25 cmH<sub>2</sub>O

Pressure drop through mask with wide Cushion at 50 L/min: 1.0 ± 0.1 cmH<sub>2</sub>O

Pressure drop through mask with wide Cushion at 100 L/min: 1.3 ± 0.25 cmH<sub>2</sub>O

### MASK DEAD SPACE

Small = 28 cc

Medium = 26 cc

Large = 28 cc

Wide = 34 cc

### SOUND

The sound power level of mask is 26.8 dBA, with uncertainty 2.5 dBA

The sound pressure level of the mask is 18.8 dBA, with uncertainty 2.5 dBA

### ADDITIONAL INFORMATION

The F&P Evora Nasal mask and accessories are not made with natural rubber latex.

For single patient reuse, the mask should be replaced no later than 12 months after the date of first use.

If any visible deterioration of a mask system component is apparent (for example cracking, discoloration, or cushion tears), the component should be discarded and replaced. Any serious incident that has occurred in relation to the use of the device must be reported to Fisher & Paykel Healthcare.

### STORAGE

Store dry mask in clean conditions out of direct sunlight. Storage temperature: -20 to 50°C (-4 to 122°F)

## WARNINGS

---

- This mask is not suitable for life support ventilation.
- This mask must be used under qualified supervision for users whom are unable to remove the mask themselves.
- Your mask should only be used with approved CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. Your mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly. To avoid having an adverse effect on the safety and quality of the therapy, the exhaust holes on the mask should never be blocked.
- **Explanation of Warning:** CPAP and Bi-Level machines are intended to be used with special masks, which have exhaust holes to allow continuous flow of air out of the mask.
  - i. Device switched on (with mask connected): When the CPAP or Bi-Level machine is turned on and functioning properly, new air from the CPAP or Bi-Level machine flushes the exhaled air out through the mask's exhaust holes.
  - ii. Device switched off (with mask connected): When the CPAP or Bi-Level machine is turned off, not enough fresh air will be provided through the mask, and exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can lead to suffocation in some circumstances.
- At low CPAP or Bi-Level pressures the flow through the exhaust holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing of exhaled air may occur.
- Discontinue use and consult your healthcare provider or physician if the following occurs:
  - i. Discomfort or irritation in or near the eyes.
  - ii. Excessive leaks between mask cushion and face.
  - iii. Gum, jaw or mouth soreness/discomfort or an aggravation of an existing dental condition occurs.
  - iv. Allergic reaction.
  - v. Mask discomfort or irritation.
- Do not use if vomiting or feeling nauseous.
- Mask use with oxygen: With your CPAP or Bi-Level machine, the oxygen flow must be turned off when the CPAP or Bi-Level machine is not operating. Oxygen accumulated in your CPAP or Bi-Level machine enclosure will create a risk of fire. You should not smoke or be in the vicinity of someone smoking while using oxygen with your mask.
- As part of good hygiene, always follow cleaning instructions and use mild soap. Some cleaning products may damage the mask, its parts and their function, or leave harmful residual vapours that could be inhaled if not rinsed thoroughly.

## CAUTIONS

---

- Use your mask for its intended use as directed in this guide.
- If any visible deterioration of a system component is apparent (cracking, discolouration, tears etc), the component should be discarded and replaced.

## WARRANTY STATEMENT

---

Fisher & Paykel Healthcare warrants that the mask (excluding foams and material diffusers), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in [www.fphcare.com/sleep-apnea/masks/warranty-mask/](http://www.fphcare.com/sleep-apnea/masks/warranty-mask/)

## CONTRAINDICATION

---

None known.

## DISPOSAL

---

Your mask does not contain any hazardous material and may be disposed of with general waste.

## PATENT INFORMATION

---

For patent information, see [www.fphcare.com/ip](http://www.fphcare.com/ip)

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